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HK Pregnancy Tester
for Sheep and Goats
Operating Instructions

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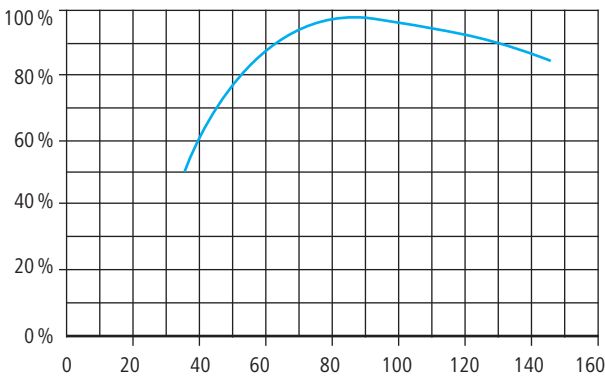
The probe should be placed in the the stifle fold in front of and above the udder on the abdominal wall and directed to the animals opposite side, between the hip-bone and the rear rip.

Anteil der richtigen Diagnose • pourcentage des diagnostics justes • percentage of correct diagnoses

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Results of a pregnancy diagnosis test from 167 goats and 122 sheep between the 35th and 120th day after mating (Reference: DLG-report no. 4621)

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Warranty

Albert Kerbl GmbH grants you an 2-year guarantee on this article in accordance with our general guarantee conditions. These general warranty conditions can be found at www.kerbl.com/guarantee. We will also be happy to send you the warranty conditions by post. Your statutory warranty rights remain unaffected by our guarantee.

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CE-/UKCA-conformity declaration

Albert KERBL GmbH hereby declares that the product / device described in these instructions complies with the fundamental requirements and other relevant stipulations and regulations. The CE-/UKCA-mark confirms compliance with the Directives of the European Union or the relevant UK legislation.

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Electronic scrap

Disposing of this device after its service life is the responsibility of the operator. Please consult the valid national regulations. The device must not be disposed of in household waste. In accordance with the stipulations of the EU Directive on the Disposal of Electrical and Electronic Devices, the device can be disposed of free of charge at the local waste collection or recycling centre. Alternatively, it can be returned to retailers who offer a collection service. The proper disposal helps to ensure environmental protection and prevents any adverse effects on human health and the environment.

Albert Kerbl GmbH

Felizenzell 9 84428 Buchbach, Germany www.rheintechnik.de
info@kerbl.com +49 8086 933-100

Kerbl UK Ltd

Lands End Way Oakham, Rutland LE15 6RF, UK
www.kerbl.co.uk