WALDHAUSEN

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- D** Gebrauchsanweisung Waldhausen Health + Care Massagegerät Pro, Hot & Cold
- E** Instructions for use for Waldhausen Health + Care Massager Pro, Hot & Cold
- F** Mode d'emploi des Waldhausen Health + Care Masseur Pro, Hot & Cold
- ES** Instrucciones de uso Waldhausen Health + Care Masajeador Pro, Hot & Cold
- IT** Istruzioni per l'uso Waldhausen Health + Care Massaggiatore Pro, Hot & Cold

Art.-Nr./Product no./N° Art.:

5700600

WALDHAUSEN
**HEALTH
+ CARE** 

www.waldhausen.com

Waldhausen GmbH & Co.KG
Von-Hünefeld-Str. 53 | 50829 Köln | Germany
info@waldhausen.com

D

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Pb = Batterie enthält Blei

Cd = Batterie enthält Cadmium

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Read these operating instructions carefully, store them for future reference, make them available to other users and follow the instructions.

WARNING

- The device is only intended for private use at home, not for commercial use.
- This device may be used by children aged 8 years and older and by persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been supervised or instructed in the safe use of the device and understand the hazards involved.
- Children must not play with the device.
- Cleaning and user maintenance must not be carried out by children unless they are supervised.
- If the power cord of this device is damaged, it must be disposed of. If it cannot be removed, the device must be disposed of.
- You must never open or repair the device under any circumstances, otherwise proper function is no longer guaranteed. Failure to comply will void the guarantee.

Dear Customer

We are very pleased that you have chosen a product from our range. Please read the operating instructions carefully and keep them in a safe place for future reference.

Kind regards,
Your Waldhausen team


CONTENTS

1. Delivery contents.....	18
2. Explanation of symbols.....	19
3. Get to know the device.....	20
4. Warning and safety information.....	20
5. Intended use.....	22
6. Device description.....	23
7. Commissioning.....	24
8. Application.....	24
9. Charging.....	26
10. Technical specifications.....	27
11. Maintenance and proper storage.....	28
12. Common problems.....	28
13. Troubleshooting.....	29
14. Disposal.....	29

1. DELIVERY CONTENTS

Check the delivery contents for external intactness of the cardboard packaging and completeness of the contents. Before use, make sure that the device and its accessories are not visibly damaged. All packaging material must be removed. If in doubt, do not use it and contact your trusted dealer.

- 1x massager
- 5x interchangeable massage heads
- 1x instruction manual
- 1x charging cable: USB-C, USB 2.0

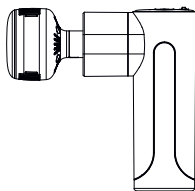Massage heads

2. EXPLANATION OF SYMBOLS

	WARNING Warning notice about risk of injury or dangers to your health.
	CAUTION Safety notice on possible damage to the device/accessories
	INFORMATION Reference to important information.
	The device has double protective insulation and therefore conforms to protection class 2.
	Read the operating instructions.
	For indoor use only.
	Dispose of packaging in an environmentally friendly manner.
	Dispose of the device in accordance with EC Directive WEEE (Waste Electrical and Electronic Equipment).
	Manufacturer
	CE MARKING This product meets the requirements of the applicable European and national guidelines.
	The products verifiably meet the requirements of the technical regulations of the EAEU.

E

3. GET TO KNOW THE DEVICE

With this electric massager, you can give your horse a soothing massage effectively and without extra help. Yourself as well, of course. Massages can have a relaxing or stimulating effect and are often used against muscle tension, pain and fatigue. The device offers a powerful, intensive massage for many parts of the horse. The device is not intended for medical or commercial use.

4. WARNING AND SAFETY INFORMATION

Please read these operating instructions carefully. Failure to observe the following information can cause injury to humans or animals, or damage to property. Store these operating instructions in a safe place and make them available to other users. Hand these operating instructions over when you pass the device on to a new owner.

WARNING

Keep children away from the packaging material. There is a danger of suffocation.

Electric shock

WARNING

As with any electrical device, this massager should be used with care and caution to avoid electrical shock hazard.

Precautions when using

- When the battery is fully charged, the charging process must be stopped.
- Do not use the device while it is charging.
- Only use the device in accordance with the operating instructions.
- Do not place the product in or near a fire source.
- Do not intentionally drop the product.
- Do not operate or charge the product without supervision.
- Due to the heat-generating surface of the device, the device must be used with care on heat-sensitive parts of the body.

In the event of defects or malfunctions, switch off the device immediately.

Make sure that the massager and the cord do not come into contact with water or other liquids.

Therefore, use the device

- only in dry interior rooms,
- only with dry hands,
- never in the bath, in the shower or with a filled sink,
- never in swimming pools, whirlpools or saunas.

Never reach for a device that has fallen into water.

REPAIR

WARNING

Repairs to electrical devices may only be carried out by specialists. Improper repairs can result in considerable danger for the user. For repairs, contact customer service or an authorized dealer.

FIRE HAZARD

WARNING

Improper use or use contrary to the instructions for use may cause a fire hazard! Therefore, operate the massager

- never longer than 15 minutes (risk of overheating) and allow the device to cool down for at least 15 minutes before using it again.
- never under a cover, e.g. B. Blanket or pillow.

Never leave the device unattended, especially when children are nearby.

HANDLING

WARNING

The device must be switched off after each use and before each cleaning.

- Make sure that the moving parts can always move freely.
- Make sure that no hair gets caught in the device.
- Avoid contact with pointed or sharp objects.
- Protect the device from heat sources that are too close by, such as ovens, radiators or direct sunlight.
- Use only with the attachments included in the scope of delivery.

5. INTENDED USE

This device is intended for massaging individual parts of the body. It cannot replace medical treatment. Do not use the massager if one or more of the following warnings apply to your pet. If you are unsure whether the massager is suitable for your horse, consult your veterinarian.


WARNING

When using the massager on your pet,


- never massage the heart area if it is suffering from heart disease and ask your veterinarian if it can be used for other areas of the body as well,
- never use it on swollen, burnt, inflamed or injured skin and body parts or open wounds,
- never use it during pregnancy,
- do not use it on the head (eyes), on the larynx or other particularly sensitive parts of the body,
- never use it while the animal is sleeping,
- never use it if it is seriously ill or after an operation,
- never use it if it has a fever, cough, runny nose, varicose veins, phlebitis, nerve diseases or acute inflammation,
- do not use it if the animal's medical condition is unclear.

The device is only intended for the purpose described in these operating instructions. The manufacturer cannot be held liable for damage caused by improper or careless use.

6. DEVICE DESCRIPTION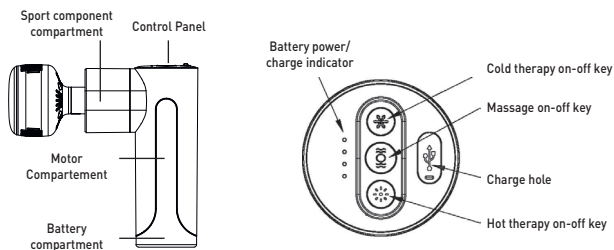

Massage heads

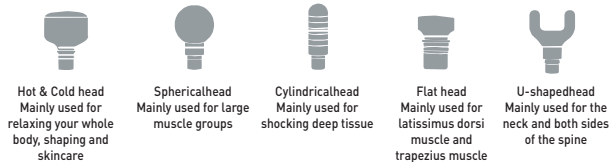7. COMMISSIONING

- Remove the packaging.
- Check the device and accessories for damage.
- Charge the device fully before using it for the first time.

8. APPLICATION

1. When the device is off and a function key is briefly pressed, the battery indicator shows the current battery level. The device will not start.
2. When the device is off: If the vibrate and tap button is pressed for three seconds, the device starts at low intensity - the indicator lights up blue.
With another short press of the button, the intensity changes to medium level - the display lights up purple.
If the button is briefly pressed again, the intensity changes to the highest level - the display lights up red.
To switch off, press the button for three seconds.
3. When the device is off: When the heat therapy on button is pressed for three seconds, the heating function will start at around 35°C and the indicator will turn blue.
After another brief press of the button, the temperature is increased to approx. 40 °C and the display turns purple.
After a second short press on the button, the temperature will increase to approx. 45 °C and the display will turn red.
To switch off, press and hold the heat therapy button for three seconds.
4. When the device is off: If the cold therapy on/off button is pressed for three seconds, the cold therapy function will start at around 20°C and the indicator will turn blue.
After another brief press of the button, the temperature changes to around 15 °C and the display turns purple.
After a second short press, the temperature is around 10 °C and the indicator lights up red.
To switch off, press the button for three seconds.
5. If the device is switched on and a function key is briefly pressed, the corresponding function is started.

6. When all functions are off, the display will show the battery power continuously for five seconds and the device will be in standby mode.
The device can be restarted by briefly pressing any function key. The device switches off automatically if it is not used for five seconds.
7. The above heat or cold therapy function needs to install the cold and hot massage head, because other massage heads don't have these functions.
8. Vibration, heat therapy and cold therapy functions can be operated independently.
9. The device will turn off automatically after 15 minutes.

What can a regular massage do for my horse?

- Reduction of tension
- Improved blood circulation → higher muscle performance
- Improving body awareness
- Increased well-being
- Strengthening of the horse-human relationship
- Stimulating the metabolism
- Activation of self-healing powers
- Relaxation after exertion

Horses can be massaged at any time

A massage **before work** can relieve tense muscles.

The horse can then use each muscle better during training and build up faster. The rider can benefit from a faster release from the horse (massage does not replace warm-up riding, it only increases muscle performance!) and improved flexibility in all gaits.

Massage **after work** can provide relaxation and less muscle soreness.

Duration of a massage: 20 - 90 minutes, depending on the horse's well-being

WARNING

The massage must be perceived as pleasant and relaxing at all times.


E

Stop the massage or change the position of the massager if you feel the massage is painful or uncomfortable for your horse.

Select/change essay

1. The disassembly and assembly of the massage head is pluggable and screwless.

Assembly instructions for Hot & Cold massage head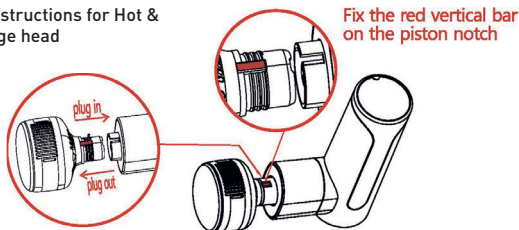


2. Fix the red vertical bar of the Hot & Cold massage head to the piston notch.

3. Make sure the attachment is fully inserted.

9. CHARGING

The charging port is located at the top of the product's lever handle.

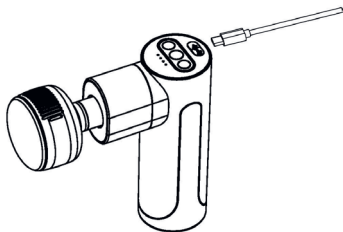1. Before first use, charge the device for about 5 hours until fully charged.
2. A special USB connection cable with TYPE-C jack is provided for the battery, the other end is equipped with any 5 volt charging adapter. (5V and 2A is recommended)
3. All of the device's battery indicators will flash regularly, indicating that the battery is charging.
4. When all battery indicators on the device stop blinking and stay on, this indicates that the battery is fully charged.
5. The battery can be charged at any time and at any charge level under certain environmental conditions.
6. Charge in time when the power is less than 25%.
7. It is not recommended to reduce the battery power to 25%.
8. The average running time is three hours after charging the device, which depends on the frequency of vibration and the pressure applied during the period of use.

10. TECHNICAL SPECIFICATIONS

Product name:	W-Health & Care Massagegerät Pro, Hot & Cold
Product model:	5700600/HXR-B028
Net weight:	620g
Input voltage:	5V=2A
Power:	25W (Max:35W)
Battery parameters:	11.1V 2400mAh
Motor Speed:	1800-2400RPM
Charging time:	about 5 hours
Temperature range:	10-45°C (±5°C)
Dimension of the machine:	145x50x150mm
Battery life:	About 2-3 hours with Tapping
	About 2-3 hours with Cooling
	About 3-4 hours with heating
Automatic timing:	15 min

11. MAINTENANCE AND PROPER STORAGE

E

Cleaning

Once the device has been unplugged and has cooled off, wipe it with a slightly damp cloth. You can use a light detergent when doing this.

WARNING

- Unplug the device before cleaning.
- Clean the device only in the manner specified. Under no circumstances should liquids enter the device or the accessories.
- Do not use the device again until it is completely dry.

Storage

Store the massager away from children in a dry, cool place. We recommend storing it in the original packaging.

12. COMMON PROBLEMS

The launch failed:

1. Excessive temperature:

If the environment where the massage gun is operated is over 40°, the massage gun may trigger overheat protection and the device may not start normally.

Solution: Keep it below 40°C for 2 hours and then start it.

2. The battery power is „0“:

In the event of a dead battery, the battery will produce a low power protection called over-discharge protection. At this moment, the massage gun can't work normally when turned on again.

Solution: Keep the shutdown state, connect a charger and put the massage gun in charging mode. Unplug the charger when fully charged. Activate at this time the on-off button to allow the massage gun to work normally.

Error while charging:

1. Overheating of the massage gun: If there is an „overttemperature“ in the main part, charging is not possible even though a charger is available.

Solution: Keep the temperature below 40°C for 2 hours and then charge again.

13. TROUBLESHOOTING

fault description

1. The massage head slips: Check whether the massage head is switched off and installed in place.
2. Check whether the power supply is sufficient (please charge the device for more than 5 hours if it is not used for a long time).
3. The device could not be connected. Check if the charging port is charged.
4. The cause is due to the setting of the protection function for long-term use.
5. The tapping function will stop switching from working mode to standby mode after the massage head is in standby mode. Restarting is possible with a short press of the locked rotor button.

E

14. DISPOSAL

ATTENTION

You must dispose of the used, completely discharged rechargeable batteries via specially marked collection containers, special waste collection points or via the electrical dealer. You are required by law to dispose of the rechargeable batteries. Remove the battery from the device by carefully opening the massager. To do this, unscrew all screws on the housing with a screwdriver and remove the housing cover. Now disconnect the battery from the device and dispose of it properly.

You will find these signs on batteries containing harmful substances:

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WALDHAUSEN GmbH & Co. KG